

Specimen Collection Reference Card



Scan QR Code
for more test information



Light Blue

D-dimer
Protrome/INR
PTT

Note: Tube must be filled to the top of the label.

Antithrombin AG
Factor VIII
Factor XI
Fibrinogen
Lupus Screen
Plasminogen
Protein C
Protein S

Note: Spin, separate into pour-off tube, and freeze.

SST (Cap color may vary)

AFP (tumor marker)
ANA Multiplex w/ Reflex
Alpha 1 Antitryp
ANTI - DNA
ANTI - Thyroid Abs
ASO Titer
Basic Metabolic Panel
CA 125
CA 15-3
CA 19-9
CCP, Ab, IgG
CEA
Ceruloplasmin
CMP
CMV IgG, CMV IgM
C3, C4
CH50 (must be frozen)
Cortisol
CRP (ultrasensitive)
DHEA-S
Digoxin
Dilantin
Electrolytes
EBV IgG, EBV IgM
ENA Abs (SSA, SSB, SM, RNP)
Estradiol
Ferritin
Folate
Free Kappa & Lambda
FSH
Hepatic Function Panel
Hep A IgG, Hep A IgM
Hep B AB Quant
Hep BS Ag
Hep B Core IgG
Hep B Core IgM
Hep C
Beta HCG

Haptoglobin
HSV-I IgG
HSV-II IgG
HIV - I/II
Homocysteine
IgA, IgE, IgG, IgM
Immunofixation
Lipid Panel
LH
Lithium
Lyme Disease Ab
Phenobarbital
Prealbumin
Progesterone
Prolactin
Protein Elect.
PSA
Free PSA
PTH-Intact
RA
RPR
Syphilis Total AB
Rubella IgG
Sars-Cov 2 IgG
Tegretol
Testosterone Free
Testosterone Total
Theophylline
TIBC/UIBC
Toxo IgG, Toxo IgM
Transferrin
T Uptake
T3, T4 Free
T3, T4 Total
T7
TSH
Valproic Acid (depakote)
Vitamin B-12
Vitamin D 25 Hydroxy

Lavender

ABO Group & RH Type
Antibody Scr. ID if Pos.
BNP
CBC w/ Diff
Hgb A1C
Hgb Electrophoresis
Lead
Retic Count
Sed Rate

Note: Tube must be minimum half full.

Grey

Glucose
Glucose Challenge - 1 Hr
Glucose Tolerance - 2 Hr
Glucose Tolerance - 3 Hr

Notes:
Tube must be minimum half full.
Write on tube(s) what time specimen was drawn.

Quantiferon TB Kit

Notes:
Order of Draw: Grey, Green, Yellow, Lavender.
Do NOT use butterfly due to limited vacuum in tubes.
Do not spin.



Questions? Call The Lab.

(248) 912-1700

Revised 1/28/2025

Order of Draw: SPS/Light Yellow, Light Blue, SST, Plain Red, Green, Lavender, Grey